

## Cleaning and storage

1. Empty the reservoir if you plan to store the device for more than 2 days.
2. If MyPurMist has been in storage for an extended period of time, fill the water reservoir with distilled water and let the device run for 5 minutes to prevent any odor.
3. MyPurMist's patented technology will not transmit germs in the mist that you breathe. However, periodically you may clean the outside of the device with a damp cloth. Make sure the device is unplugged while cleaning the outside.
4. Do not put in dishwasher or immerse in water. Do not use abrasives to clean.
5. You may periodically remove the mask from the device and clean with soapy water.
6. Make sure to store both the device and water at room temperature.

## Troubleshooting and contacting us

When troubleshooting your device, please make sure that you have distilled water in the device and that there is a green light on the power supply.

- A red light means the unit is out of water. Refill reservoir.
- A green light means the unit has water and is ready. Press start.
- If you hear the fan during a red or green light, the drying cycle is on. Leave it until it stops by itself or start another session.
- A red blinking light means you are using water that is not distilled. Please replace water with MyPurFill or distilled water. You may have to do this repeatedly until the red blinking light changes to green.

**If any blue lights are blinking please contact us. We are here to help!**

**For warranty claims do not return it to the store or online merchant.**

Email: Care@MyPurMist.com

Web: www.MyPurMist.com

Phone: 1-855-697-8764 (general customer support)

1-866-925-6710 (warranty & tech support)

Vapore, LLC, 1130 Burnett Ave. Ste. P, Concord, CA 94520

Model: MyPurMist OS2 or OS3

Do not attempt to fix the device yourself. Contact us if it needs repair or replacement.

## Warranty

MyPurMist comes with a limited warranty against defective material or workmanship for a period of one year from the date of purchase. Any defective part will be repaired or replaced at no charge if the device has not been tampered with and has been used according to this user manual. It is especially important that you only use distilled water with your device. Retain your receipt to prove warranty. This warranty gives you specific legal rights, as well as other rights that may vary from state to state.

[Please register your device on mypurmist.com.](http://mypurmist.com)

*MyPurMist shown with optional small mask*



## For your safety

**PLEASE READ ALL INSTRUCTIONS**

When using electrical appliances, basic safety precautions should always be followed, including the following:

- Use MyPurMist (device) only for its intended use as described in this manual.
- To protect against a risk of electric shock, do not use device if it is wet or immersed in water or other liquid. Do not use or store device where it may fall in water.
- The device should always be turned "OFF" and be silent (i.e. let the automatic drying cycle stop the fan) before unplugging from outlet.
- Do not operate device with a damaged cord or if the device has been damaged.
- Keep device and cord away from heated surfaces. Position cord to avoid pulling or tripping.
- Always unplug the device when not in use.
- Use MyPurFill water or distilled water in the device. Do not overfill. **Using any additives in the water reservoir, any other liquid or water with minerals (e.g. bottled, purified or boiled water) will damage the device and will void the warranty.**
- Do not use device without the water reservoir cap in place.
- The device is **not recommended for children under the age of 13.**
- Only use MyPurMist accessories.
- Do not use device near anything flammable.
- During use do not block the air vents on the sides or bottom of the device such as wrapping the product in a blanket.
- Never insert any object or body part into any opening.

## SPECIAL INSTRUCTIONS

- Plug: This device will fit into a polarized outlet either way. If it does not fit your outlet, contact a qualified electrician. Do not attempt to modify the plug in any way.
- Extension Cords: If an extension cord is necessary, use an approved extension cord. Care should be taken to arrange the cord so that it cannot be pulled or tripped over.

**This device complies with Part 15 of the FCC Rules.** Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**MyPurMist**<sup>®</sup>  
handheld steam inhaler

Effective

Natural

Easy

Advanced

User manual



# How to use MyPurMist

1. Plug the device into a standard wall outlet. You will see a green light in the power supply (box).

2. Attach mask by tilting it forward and catching the lower lip of the mask under the tab in the top opening. Pull back until it clicks in. Do not push the mask straight down (figure 1).



Figure 1

3. Fill water reservoir with MyPurFill water or distilled water. Place cap over reservoir. Let water soak in for 30 seconds. For subsequent uses refill water before each use. If some water remains after a session, just leave it there for the next session. Discard the water if you plan on not using MyPurMist for two or more days.

4. Press the ON/OFF button for instant steam. A blue LED comes on and steam starts flowing right away. There is no waiting period. Enjoy! You can start and stop the device as much as you want - just make sure you let it run the drying cycle once finished (see #9).

5. Use the temperature button to vary the temperature between approximately 100°F (one blue LED) to approximately 115°F (four blue LEDs).

6. Use MyPurMist by placing mask over your nose and mouth lightly. Breathe comfortably through your nose or mouth. Use comfortably at any angle.

7. MyPurMist turns off automatically when out of water (a red light appears). For longer use, refill the reservoir (a green light appears) and turn the device on again. If the red light is blinking make sure you are using correct water. Replace water with MyPurFill or distilled water. Once you see a green light the device is ready to start again.

8. Please note that using any additives in the water reservoir, any other liquid or water with minerals (e.g. bottled, purified, or boiled water) will damage the device and will void the warranty.

9. **Automatic Drying Cycle:** Each time after the device is turned OFF, a fan will run for 10 min to dry all the internal parts of the device. It stops automatically. This ensures germ-free operation. Do not unplug the device when the fan is running. Just leave it to take care of itself.



Medical-grade latex-free mask

LED lights for temperature and operational status (4 blue and 1 red/green LED)

Temperature adjustment 100° to 115° F

ON / OFF button

Water reservoir. Uses approximately one ounce per 25 minutes

# When to use MyPurMist

MyPurMist can be used at any time and for any length of time.

MyPurMist reinvents steam therapy utilizing breakthrough patented “CFV” hospital technology\*. MyPurMist sets a new standard for safety, convenience, speed, and ease of use.

Steam or humidification may provide symptomatic relief from allergies, sinusitis, nasal, sinus and chest congestion, colds, flu, laryngitis, coughs, parched throat, lessening mucus production and discomfort from sore throat.

*“Breathing warm, moist air (natural steam) provides immediate relief by lessening mucus production, nasal congestion, coughing, sore throat discomfort and headache pain.”*  
—Prevention Magazine 12/89

Use MyPurMist as often as needed. Suggested use is for 10-15 minutes two times per day - including before sleeping. Best results are typically obtained after 1-2 weeks of regular use. Use while watching TV, relaxing with a book, at work or in bed before sleeping.

**Consult your doctor.**

MyPurMist can be used anywhere at home or office or while traveling - even overseas.

MyPurMist accepts power outlets ranging from 100V to 240V and 50 or 60 Hz.

MyPurMist is great for the whole family including children 13 years and older.



# MyPurMist accessories improve your comfort and pleasure

## Adult and Small Size Mask

Medical-grade and latex-free extra mask for MyPurMist. (One mask)



## ScentPad Eucalyptus & Peppermint

Soothing aromatic Eucalyptus and fresh Peppermint vapors for non-medicated comfort.



## MyPurFill Water

Pure distilled water in convenient 12 oz. bottles for MyPurMist. (Case of 12 bottles)



## Hands-free Holder

This lightweight holder lets you use your MyPurMist inhaler hands-free.



Available at selected retailers. See [www.MyPurMist.com](http://www.MyPurMist.com) for retail locations in your area and online. Also available by calling 1-855-697-8764 or at [www.MyPurMist.com](http://www.MyPurMist.com)