

# MyPurMist Quickstart Guide

1. Attach mask (insert at a 45° angle - see manual).
2. Fill water reservoir with MyPurFill water or distilled water. Place cap over reservoir. Let water soak in for 30 seconds. Plug the unit in.
3. Push the ON/OFF button: Enjoy the benefits of MyPurMist!

---

## Useful hints

Each full reservoir provides approximately 25 minutes of steam therapy.

Adjust the temperature to your comfort using the rocker button. There are four blue lights for temperature (approximately 100 to 115°F).

When done, push the ON/OFF button. A green light will come on and the fan will run for 10 minutes to dry the internal parts (do not unplug). No cleaning required.

If a RED light comes on, the device is out of water. You can either just leave the unit to runs its drying cycle or refill with water and continue.

## Lights

A red light means the unit is out of water.

A green light means the unit has water and is ready.

If you hear the fan during a red or green light, the drying cycle is on.

A red blinking light means you are using water that is not distilled. Please replace water with MyPurFill or distilled water. You may have to do this repeatedly until the red blinking light changes to green.



For general support please email [care@mypurmist.com](mailto:care@mypurmist.com)  
For technical support please email [techsupport@mypurmist.com](mailto:techsupport@mypurmist.com).  
Do not return it to the store or online merchant.



Secure your warranty today by registering your MyPurMist inhaler.  
Go to [www.MyPurMist.com/register](http://www.MyPurMist.com/register)

